

# deconex® 64 NEUTRADRY


**Neutral rinsing and drying aid,  
biocompatible**

For reprocessing medical instruments and utensils  
in washer-disinfectors (WD)



## Application

deconex 64 NEUTRADRY is used as a rinsing and drying aid in washing machines for medical instruments and utensils. It is dosed into the final rinsing water, which usually is also used for thermal disinfection (80-93 °C).

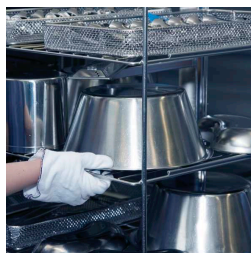

## Properties

deconex 64 NEUTRADRY

- lowers the surface tension of water
- shortens the drying time by up to 50%
- is biocompatible and correspondingly certified
- is pH-neutral
- has a slightly lubricating effect (on joints)

## Ingredients

Surfactants, organic acids, preservative

## Dosage

0.3-1.0 ml/l (depending on the water quality)

## Information on use

Deionized water should always be used for thermal disinfection/final rinsing.

## Material compatibility

The product features a broad material compatibility. However, the following has to be considered:

The use of a drying/rinsing aid leaves traces of surfactants on surfaces. These surfactants in combination with steam sterilization can lead to stress cracking in certain plastic structures under tension.

Known examples are the PPSU lids of the PrimeLine containers of Aesculap. For this reason, Aesculap explicitly discourages users from using drying/rinsing aids for these container lids.

## Chemical/physical data

pH	1% solution in demineralized water	approx. 7.9
Density	concentrate	1.0 g/ml
Appearance	concentrate	transparent, colourless to yellowish liquid

# deconex® 64 NEUTRADRY

## Availability

Please ask your local representative about current container sizes.

Containers, screw caps and labels are made of recyclable polyethylene.

## Additional information

For information concerning safety at work, storage and waste disposal/effluent, please consult the corresponding safety data sheet.

Take advantage of our vast know-how! Please, contact us for further information regarding your specific application.



11924 Forest Hill Blvd., Suite 22-162, Wellington, FL 33414  
Tel. +1 561 784 56 43, Fax. +1 561 880 68 97  
[www.racoma.com](http://www.racoma.com), [contact@racoma.com](mailto:contact@racoma.com)

## Manufacturer:

### Borer Chemie AG

Gewerbestrasse 13, 4528 Zuchwil / Switzerland  
Tel +41 32 686 56 00 Fax +41 32 686 56 90  
[office@borer.ch](mailto:office@borer.ch), [www.borer.ch](http://www.borer.ch)

All information provided is based on our current knowledge and it does not constitute a legally binding assurance of specific product properties.